



ANTIBACTERIAL DRESSING

MEDIHONEY®

Antibacterial Honey Apinate™ Dressing

FEATURES

- Antibacterial *Leptospermum* (Manuka) Honey
- Calcium alginate fabric
- Protects delicate tissue

INDICATIONS

- Leg/foot ulcers
- Pressure ulcers
- Infected wounds
- Sloughy wounds
- Malodorous wounds
- Donor and recipient graft sites
- Burns
- Surgical wounds
- Diabetic wounds
- Abrasions

SUPERIOR WOUND BED PREPARATION

Medihoney® Antibacterial Honey Apinate™ Dressing

protects the wound by creating a barrier against wound pathogens, including antibiotic resistant strains, and therefore reducing the risk of infection. The osmotic action produces an outflow of body fluid which assists the removal of wound bacteria, endotoxins, debris and slough, providing a cleaner wound, rapidly removing malodour and helping to reduce the inflammatory response, oedema and exudate levels. Granulation and epithelialisation are enhanced through provision of the optimal healing environment.

MEDIHONEY® Antibacterial Honey Apinate™ Dressing

creates a micro-environment that supports healing and provides an antibacterial barrier⁹.



international.dermasciences.com

MEDIHONEY® is a trademark of Medihoney Pty Ltd. and licensed by Derma Sciences, Inc.



Derma Sciences Europe Ltd.
Tel: +44 (0)1628 625916
Fax: +44(0)1628 671112
Contact: orders@dermascienceseurope.co.uk
international.dermasciences.com



ANTIBACTERIAL DRESSING

MEDIHONEY®

Antibacterial Honey Apinate™ Dressing



DIRECTIONS FOR USE

1. Protect vulnerable wound edges with a suitable barrier cream such as MEDIHONEY® Barrier Cream.
2. Place the dressing onto the wound surface ensuring that the dressing is in contact with the wound bed. Dressings can be placed side by side to cover large areas. MEDIHONEY® Antibacterial Honey Apinate™ Dressing should be cut with sterile scissors or folded to fit the shape of the wound. Alternatively the wound may be filled with MEDIHONEY® Antibacterial Medical Honey™ before applying MEDIHONEY® Antibacterial Honey Apinate™ Dressing.
3. A suitable non-adherent sterile pad or dressing should be applied to cover the MEDIHONEY® Antibacterial Honey Apinate™ Dressing. The dressing should be sufficiently absorbent to manage wound exudate.
4. The wound may be cleaned between dressing changes in accordance with standard wound cleaning protocol.

Frequency of change: The MEDIHONEY® Antibacterial Honey Apinate™ Dressing requires changing when the dressing has been diluted by wound exudate. The dressing may be left in place for up to 7 days depending on wound exudate. If maceration of the surrounding skin occurs more frequent changing of the dressing is required.

ITEM#	PRODUCT NAME
793	1.9cm x 30cm, Rope Dressing
794	5cm x 5cm Dressing
795	10cm x 10cm Dressing